

Patient Cable

BJ-961D, BJ-962D

General

The patient cable of BJ-961D, BJ-962D needs to be used with the electrocardiograph of Nihon Kohden or Shanghai Kohden.

Safety Information

- ⚠ DANGER:** A danger alerts the user to a hazardous situation which causes death or serious injury.
- ⚠ WARNING:** A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.
- ⚠ CAUTION:** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

- When performing MRI test or defibrillation, remove all cables from the patient. Otherwise, it may cause skin burn on the patient.
- All patient cables are only available for electrocardiograph and electrodes on the market produced by Nihon Kohden and Shanghai Kohden
- Only use with the specified electrocardiograph. Otherwise, it may result in electrical shock, and the safety and performance of the patient cables and electrodes cannot be guaranteed.

⚠ CAUTION

Do not contact with water or medicinal liquid and avoid placing in an wet area when using it.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer

SHANGHAI KOHDEN MEDICAL
ELECTRONIC INSTRUMENT CORP.
No.567 Huancheng Bei Road, Shanghai Comprehensive
Industrial Development Zone, Fengxian District,
Shanghai 201401, China
Phone +86 21-5743-6998 Fax +86 21-5743-6939
<https://www.nihonkohden.com/>



European Representative

NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, D-61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599