

ECG Connection Cord

JC-933P

General

The JC-933P ECG connection cord connects a disposable radiolucent electrode (X-transrode) to a transmitter.

Safety Information

⚠ WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

- Do not take this cord into the MRI test room. This cord is not designed to be used during MRI tests.
- Before defibrillation, all persons must keep clear of the bed and must not touch the patient or any equipment connected to the patient. Failure to follow this warning may cause electrical shock or injury.

⚠ CAUTION

- Connect the cord only to the specified instrument and follow the specified procedure. Otherwise, safety as a medical device cannot be guaranteed.
- Do not pull the cable when connected to a transmitter. Do not get a peripheral device caught on the cable. This causes instruments to break and may injure the patient or operator, and the connector might come off so monitoring is not possible.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

EC REP European Representative
NIHON KOHDEN EUROPE GmbH
Raiffenstrasse 10, D-61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599