

EEG Electrode (Collodion)

NE-133A

General

The collodion electrode is designed for EEG examination.

Safety Information

- ⚠ WARNING** A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.
- ⚠ CAUTION** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

When performing MRI test, remove all electrodes from the patient. Failure to follow this warning may cause skin burn on the patient. For details, refer to the MRI manual.

⚠ WARNING

Keep collodion and acetone away from fire because they are flammable.

⚠ WARNING

Never allow collodion or acetone to get in the patient's eyes. If collodion or acetone accidentally gets in the eyes, immediately and thoroughly wash eyes with clean water and perform medical treatment immediately.

⚠ CAUTION

Do not attach the electrode on a wound or inflamed skin.

⚠ CAUTION

Never use the electrode for electric stimulation.

⚠ CAUTION

Do not store this electrode in an atmosphere with sulfur. It will deteriorate the electrode.

⚠ CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

EC REP European Representative
NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, D-61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599