

Recorder Module

WS-752P

General

The WS-752P recorder module is for the SVM-7500 series bedside monitor. The recorder module can record three waveform traces and review data such as vital sign data and trendgraph. For details on how to use the recorder module, refer to the operator's manual and user's guide provided with the SVM-7500 series bedside monitor.

Thoroughly read this manual and the operator's manual of the connected monitor before use.

Safety Information

- ⚠ DANGER:** A danger alerts the user to a hazardous situation which causes death or serious injury.
- ⚠ WARNING:** A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.
- ⚠ CAUTION:** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ CAUTION

Turn off the bedside monitor power and disconnect the AC power cord from the AC outlet before starting installation. Installing the module with the bedside monitor power on may cause the operator to receive electrical shock.

⚠ CAUTION

Do not touch the thermal head inside the recorder module. The thermal head may be damaged by static electricity or become dirty and cause printing failure.

⚠ CAUTION

- Do not touch the recording head with any hard object. When the head is tapped with hard object, the head may crack and the heater element wire may break.
- Clean the head surface with the provided head cleaner pen before loading new paper. After a period of usage, paper dust may accumulate between the paper and the head surface and good printing cannot be obtained.
- Be careful not to cut yourself on the paper cutter in the recorder.

NOTE • When setting the recorder module, contact your Nihon Kohden representative.

- Use a wrist ground strap or wear rubber gloves to prevent electrostatic discharge. The components in the recorder module may be damaged by electrostatic discharge.

NOTE: When connecting the connection cable to a socket, insert it straight without bending or tilting it and connect it slowly and firmly without too much force. If you tilt the connector when inserting it, the connector pin may break and units cannot communicate with each other.

NOTE: Press the two tabs slightly downward, then push the recorder into the recorder magazine.

NOTE: If the recorder is not parallel to the monitor case, the recorder may get stuck in the recorder magazine. If it gets stuck, press the release lever of the recorder with a strong force as shown in the picture to take out the recorder, and then reinstall it.

Disposal

⚠ CAUTION

Dispose of Shanghai Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer

SHANGHAI KOHDEN MEDICAL
ELECTRONIC INSTRUMENT CORP.
No.567 Huan Cheng Bei Road, Shanghai Comprehensive
Industrial Development Zone, Fengxian District,
Shanghai 201401, China
Phone +86 21-5743-6998 Fax +86 21-5743-6939
<https://www.nihonkohden.com/>



European Representative

NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, D-61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599