

NMT Module AF-101P

Ideal performance for muscle relaxation monitoring

- *Visible and tangible benefits in monitoring patients*
- *Enables moderate doses of muscle relaxant and antagonist*
- *Helps confirm safe intubation and extubation*



Fighting Disease with Electronics

 **NIHON KOHDEN**

Flexible operation and installation

A handy controller with 4 hard keys gives one-touch access to the most frequent operations. The controller lets you operate the NMT module from wherever you are.

Cable management is easy even when the patient position is changed.

You can mount the NMT module anywhere by using the clip holder. Durable cables and IPX4 waterproofing help prevent trouble during operation.


Supports better process for better outcome

Preparation is easy. Just plug the smart cable into the Life Scope monitor MULTI socket. There are only 2 sensors and 2 electrodes.

You can do calibration near the patient by using the CAL key on the hand-held controller.

You can operate 5 stimulation modes: TOF (Train Of Four), PTC (Post Tetanic Count), Single, DBS (Double Burst Stimulation), and TET (Tetanic).

The NMT module can help you achieve better outcomes.

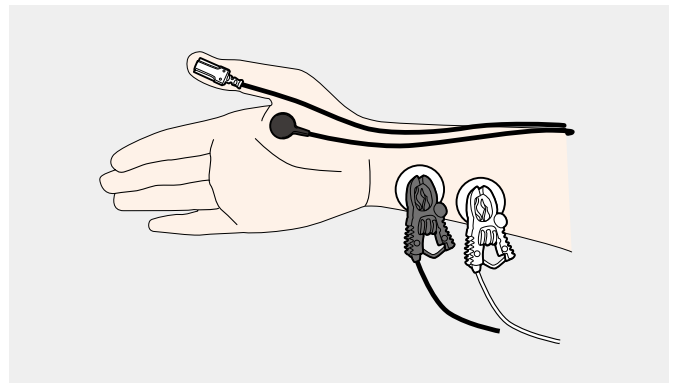

Standard items

1	Main cable	JW-101P K942A
2	Stimulation cable	BF-101P K942B
3	Acceleration transducer	TA-101P P036A
4	Temperature sensor	TT-101P P036B

Optional items

5	Hand adapter	YS-114P1 Y862B
6	Acceleration transducer adapter for thumb and toe	YS-114P2 Y862C
7	Orbicularis oculi muscle acceleration transducer adapter	YS-114P3 Y862D
8	Holder	DH-101P Y862A

Compatibility

Nihon Kohden Life Scope series bedside monitors

Ask your sales representatives for details.

This brochure may be revised or replaced by Nihon Kohden at any time without notice.



NIHON KOHDEN CORPORATION

1-31-4 Nishiochiai, Shinjuku-ku, Tokyo 161-8560, Japan

Phone +81 (3) 5996-8036 Fax +81 (3) 5996-8100

www.nihonkohden.com