

NKB-301V Battery Pack

YZ-024H9

General

The NKB-301V battery pack is a rechargeable battery for TEC-5500 series, TEC-5600 series, TEC-7600 series and TEC-7700 series defibrillators and ECG-1350 and ECG-2300 series electrocardiographs.

Safety Information

⚠ WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

Do not do the following to the battery pack. It may cause leakage, overheating, explosion and fire.

- Do not short-circuit the + and – terminals on the battery pack.
- Do not put the battery pack into fire or heat the battery pack.
- Do not disassemble or modify the battery pack.
- Do not give strong impact to or deform the battery pack.
- Do not use the battery pack on unspecified instruments.
- Do not charge the battery pack on unspecified instruments.
- Do not install the battery pack with the wrong polarity.
- Do not immerse the battery pack in water or sea water or get it wet.

⚠ WARNING

Do not use or leave the battery pack in direct sunlight or high temperatures such as in a car in hot weather or in front of a stove. It may cause leakage, overheating, explosion and fire of the battery pack and degrade the performance and lifetime of the battery pack.

⚠ WARNING

- Test the battery pack once a month.
- When you start using a new battery pack, write the date of first use on the label.
- Replace the battery pack every year.
- If defibrillation is necessary during the battery test, operate the defibrillator on AC power or use another defibrillator. Do not use battery power because the battery pack may have been deteriorated or discharged.

⚠ WARNING

If the battery pack is damaged and the substance inside the battery pack contacts the skin, eyes or mouth, do not rub it. Wash immediately and thoroughly with water and see a physician.

⚠ WARNING

If the battery pack is left unused for a long period (about 1 year), it may cause overheating and leakage of the battery pack.

⚠ CAUTION

To keep the battery pack fully charged, always keep the power cord connected to the AC outlet even when the defibrillator or electrocardiograph is not used. Otherwise, the battery pack may discharge and become unusable.

⚠ CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>



European Representative
NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, 61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 27 May 2019
2nd Edition: 09 Feb 2024

