

External Paddles Holder

DP-101V

General

The DP-101V external paddles holder is a resistor used for high-voltage discharge tests when using the ND-831V external paddles with the EMS-1052 defibrillator.

Safety Information

⚠ WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

When performing energy discharge check, keep the external paddles in the external paddles holder. Do not discharge with the external paddles in the air or touching the patient. Failure to follow this warning may cause electrical shock and damage to the defibrillator.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

EC REP European Representative
NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, 61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 02 Apr 2019
2nd Edition: 03 Apr 2024

