



Battery Pack SB-950P

General

The SB-950P battery pack (hereafter referred to as “the battery pack”) is a dedicated battery for the following devices.

- CSM-1501 or CSM-1502 bedside monitor
- BSM-5700 series bedside monitor
- TEC-1000 series defibrillator

Safety Information

- ⚠ WARNING**
- A warning alerts the user to a hazardous situation which causes death or serious injury. (Double outline with thick and thin lines)
 - A warning alerts the user to possible injury or death associated with the use or misuse of the instrument. (Single thick outline)
- ⚠ CAUTION**
- A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

Failure to observe any of the following may cause battery pack malfunction, overheating, explosion, fire and skin burn.

- Do not immerse the battery pack in liquid or get it wet.
- Do not leave the battery pack near a heat source such as a stove.
- Do not charge the battery pack on unspecified instruments.
- Do not charge the battery pack in conditions outside the specified environment. (Over 45°C (113°F))
- Do not put the battery pack into fire or heat it.
- Do not short-circuit the + and – terminals on the battery pack.
- Do not give strong impact to or deform the battery pack.
- Do not disassemble or modify the battery pack.
- Do not charge the battery pack in a high temperature (over 80°C (176°F)) place such as near a stove or in sun-heated cars.

⚠ WARNING

If the battery pack is damaged and the substance inside the battery pack contacts the eyes, wash immediately and thoroughly with water and see a physician. Never rub your eyes, because you may lose your eyesight.

⚠ WARNING

Remove the battery pack from the monitor when it is not used for a long time. Otherwise the battery may leak.

⚠ WARNING

Never use a battery pack which is damaged, discolored or has leakage. It may cause overheating, explosion or fire.

⚠ CAUTION

Do not leave the battery pack within reach of the patient.

⚠ CAUTION

Do not leave the battery pack in direct sunlight or high temperatures such as in a sun-heated car. This may cause leakage of the battery pack, degrade its performance and make it unusable.

⚠ CAUTION

If the battery pack is damaged and the substance inside the battery pack contacts the skin or cloth, wash immediately with clear water. The skin may get irritated.

⚠ CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer

NIHON KOHDEN CORPORATION

1-31-4 Nishiochiai, Shinjuku-ku,

Tokyo 161-8560, Japan

Phone +81 3-5996-8041

<https://www.nihonkohden.com/>



European Representative

NIHON KOHDEN EUROPE GmbH

Raiffeisenstrasse 10, 61191 Rosbach, Germany

Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 27 Nov 2023

2nd Edition: 22 Oct 2024

