



Battery Pack

SB-121A Op.No.01

General

The SB-121A battery pack is a rechargeable battery for a ZB-120A telemetry unit or SB-120A extension unit which are used with a WEE-1200 wireless input unit.

Safety Information

- ⚠ DANGER** A danger alerts the user to a hazardous situation which causes death or serious injury.
- ⚠ CAUTION** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ DANGER

Failure to observe any of the following may cause battery pack malfunction, overheating, explosion or fire.

- Do not immerse the battery pack in liquid or get it wet.
- Do not leave the battery pack near a heat source such as a stove.
- Do not charge the battery pack on unspecified instruments.
- Do not charge the battery pack in conditions outside the specified environment (over 35°C (95°F)).
- Do not put the battery pack into fire or heat it.
- Do not short-circuit the + and – terminals on the battery pack.
- Do not give strong impact to or deform the battery pack.
- Do not install the battery pack in unspecified instruments.
- Do not disassemble or modify the battery pack.
- Do not charge the battery pack in a high temperature place such as near a stove or in sun-heated cars.

⚠ DANGER

If the liquid inside a battery pack contacts the eyes or mouth, do not rub the eyes or mouth. Wash with clean water and see a physician immediately.

⚠ CAUTION

Do not leave the battery pack within reach of the patient.

⚠ CAUTION

Do not leave the battery pack in direct sunlight or high temperatures such as in a sun-heated car. This may cause leakage of the battery pack, degrade its performance and make it unusable.

⚠ CAUTION

If the battery pack is damaged and the substance inside the battery pack contacts the skin or cloth, wash immediately with clean water. The skin may get irritated.

⚠ CAUTION

Charge the extension unit outside the patient environment. Failure to follow this caution may cause skin burn on the patient.

⚠ CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer

NIHON KOHDEN CORPORATION
1-31-4 Nishiuchi, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>



European Representative

NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, 61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599
UK Responsible Person
NIHON KOHDEN UK LTD.