

Interface

QF-911P

General

The QF-911P interface is for displaying the waveforms and data acquired from the external instrument on the bedside monitor screen.

Safety Information

⚠ WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

Connect only the specified instrument to the interface and follow the specified procedure. Failure to follow this warning may result in electrical shock or injury to the patient and operator.

⚠ WARNING

When several medical instruments are used together, ground all instruments to the same one-point ground. Any potential difference between instruments may cause electrical shock to the patient and operator.

⚠ WARNING

The PiCCO alarms do not function on the bedside monitor. When the PiCCO data on the bedside monitor is abnormal, check the alarm on the PiCCO monitor.

⚠ WARNING

Before connecting or disconnecting instruments, make sure that each instrument is turned off and the power cord is disconnected from the AC socket. Otherwise it could cause electrical shock to the patient or operator, malfunction or instrument failure.

⚠ CAUTION

Connect the PiCCO monitor to the bedside monitor using only the QF-911P interface. Do not connect the instruments using any other interface or cable. Otherwise, the measurement accuracy becomes out of range.

⚠ CAUTION

Do not get the connectors wet with water or chemical solution. Do not use the interface when connectors are wet. Correct measurement data might not be obtained.

⚠ CAUTION

- Do not pull or bend the cables with too much force. The cable may be damaged.
- Keep the interface and cable out of the way. Otherwise people may trip over it, the bedside monitor may fall and injure the patient and operator, and monitoring may be affected.

⚠ CAUTION

Never disassemble or repair the interface. If there is any problem with the interface, contact your Nihon Kohden representative.

⚠ CAUTION

Make sure that the screws on the connector are firmly fastened to the socket. Do not touch the connector pins. The operator may receive electrical shock.

⚠ CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

⚠ CAUTION

Periodically check that the interface is properly connected to the bedside monitor and the PiCCO monitor. Otherwise, the patient or operator may receive electrical shock when a metal part on a disconnected cable is touched.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>



European Representative
NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, 61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 22 Apr 2019
2nd Edition: 27 Jun 2023

