CHM cleaning kit YZ-006B0

English

General

Intended Purpose

For in vitro diagnostic use only.

CHM cleaning kit is used for in vitro diagnosis as a hypochlorous acid-based detergent for the purpose of removing blood stains in order to clean the flow path of Nihon Kohden clinical chemistry analyzers. This device is intended for laboratory professional use only.

Read the analyzer operator's manual together with this manual before and during use.

- NOTE**
- Use the CHM cleaning kit with the specified analyzers only.
 - Read the SDS (Safety Data Sheet) carefully before use. The SDS is available from your Nihon Kohden representative.

Devices Intended for Use in Combination with
CHM-4100 clinical chemistry analyzer

Materials Provided and Materials Required

Materials Provided

- MK-820W CLEANAC•820
- CHM cleaning tube

Materials Required (Not Provided with the CHM cleaning kit)

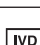
None

Intended Users

For laboratory professional use only, in laboratories with suitable equipment for clinical chemistry testing. Qualified personnel, e.g. laboratory technicians trained in clinical chemistry analysis techniques, will be able to use according to the Operator's Manual.

Symbols

The following symbols are used with the CHM cleaning kit. The descriptions of each symbol are given in the table below.

Symbol	Description	Symbol	Description
	Use by		Corrosion ¹
	Lot number		Unique device identifier
	Catalogue number		Model number
	Keep away from sunlight		Manufacturer
	Temperature limits		Authorized representative
	Caution		The CE mark is a protected conformity mark of the European Union.
	Operator's manual; operating instructions		Importer
	In vitro diagnostic medical device		

¹ The label is affixed in accordance with GHS requirements. For details, refer to "Hazards Identification" (p. 2).

Safety Information

-  **DANGER** A danger is used to alert the user to a hazardous situation which will cause death or serious injury.
-  **CAUTION** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in this operator's manual.

 **DANGER**

If the MK-820W CLEANAC•820 detergent contacts the eyes, wash immediately with plenty of water for at least 15 minutes and see a physician. The detergent can cause blindness.

⚠ CAUTION

- Do not allow the MK-820W CLEANAC•820 detergent to come into contact with acid. Contact with acid can release poisonous chlorine gas.
- Do not inhale the detergent. If inhaled, move to fresh air, rest in a posture which allows easy breathing and see a physician.
- Do not swallow the detergent. If swallowed, rinse the mouth immediately. See a physician.
- If the detergent contacts the skin, wash thoroughly and immediately with water and see a physician.
- Wear rubber gloves or goggles to protect yourself when handling the detergent.

NOTE: Do not use the CHM cleaning kit if it is past the expiration date on the label or if the CHM cleaning kit is stored under unspecified conditions.

Hazards Identification

Signal Word

Danger

Hazard Statement

- H290 May be corrosive to metals
- H318 Causes serious eye damage
- H412 Harmful to aquatic life with long lasting effects
- EUH210
Safety data sheet available on request

Precautionary Statement Prevention

- P234 Keep only in original packaging.
- P273 Avoid release to the environment.
- P280 Wear protective gloves/eye protection/face protection.

Precautionary Statement Response

- P305+P351+P338
IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P310 Immediately call a POISON CENTER/doctor.
- P390 Absorb spillage to prevent material-damage.

Precautionary Statement Storage

- P406 Store in a corrosion resistant container with a resistant inner liner.

Precautionary Statement Disposal

- P501 Dispose of contents/container in accordance with local and national regulations.

Sodium hypochlorite 1.5%

Using the CHM cleaning kit

Refer to the analyzer operator's manual.

- NOTE
- After use, store the cleaning kit in a resealable pouch.
 - Replace the detergent bottle and do not refill it. If it is refilled, the cleaning effect will decrease.

Technical Information

Composition

This product is a cleaning solution for clinical chemical analyzers containing Sodium hypochlorite as the main component.

- Sodium hypochlorite (NaClO): ≤1.5%

Sterilization Method

The CHM cleaning kit is not intended to be sterilized or kept in a sterile environment.

Environmental Conditions

Storage Environment

Temperature: 2 to 8°C (36 to 46°F)

NOTE: Store the MK-820W CLEANAC•820 detergent in a resealable pouch (shading).

Transport Environment

Temperature: 1 to 30°C (34 to 86°F)

Usage Environment

Temperature: 15 to 30°C (59 to 86°F)

Expiration Date

The expiration date is shown on the label.

Package and Catalog Number

Model	Qty	Catalog Number
YZ-006B0	MK-820W CLEANAC•820 detergent: 15 mL × 1 CHM cleaning tube: 1	T438W

Disposal

When disposing of the CHM cleaning kit, such as when the expiration date is past, follow the instructions on the SDS of the CHM cleaning kit.

NOTE: Dispose of the CHM cleaning kit according to your local laws and your facility's guidelines for waste disposal (for incineration, melt treatment, sterilization and disinfection). Otherwise, it may affect the environment.

Revision History

Edition	Date	Details	Code Number
1st Edition	04 Aug 2016	Initial issue	0614-907575
3rd Edition	27 Apr 2023	Symbols added	0614-907575C
4th Edition	06 Mar 2026	Symbols and notes are added	0614-907575D

NOTE: Changes made in the most recent edition are indicated by a bar in the left margin of each page.

Note for users in the territory of the EEA and Switzerland:
Any serious incident that has occurred in relation to the device must be reported to the European Representative designated by the manufacturer and the Competent Authority of the Member State of the EEA and Switzerland in which the user and/or patient is established.

Copyright Notice
The entire contents of this manual are copyrighted by Nihon Kohden. All rights are reserved.

 Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

 European Representative
NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, 61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599
UK Responsible Person
NIHON KOHDEN UK LTD.
Unit 3, Heyworth Business Park,
Old Portsmouth Road, Peasmarsh,
Guildford, Surrey, GU3 1AF, UK
Phone +44 14-8333-1328

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